


5-Lead Holter Hook-Up

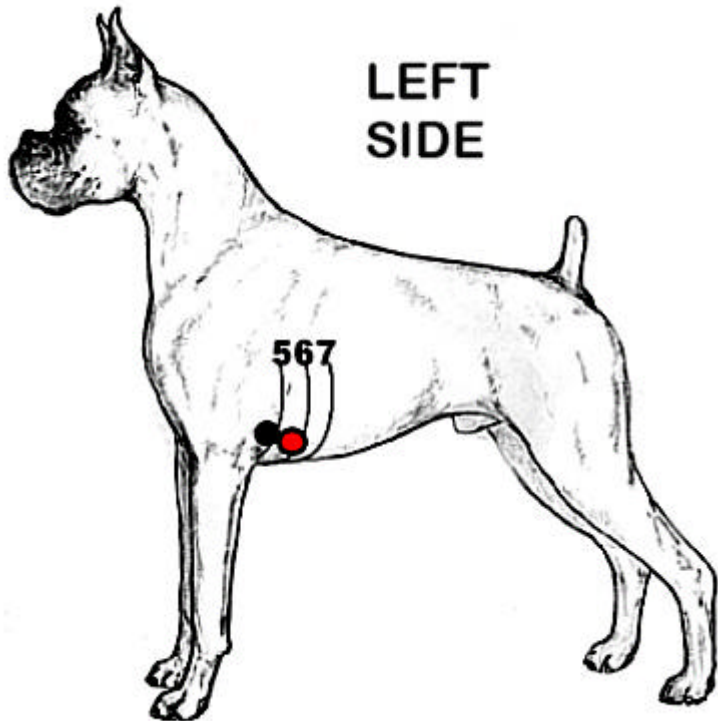


BLACK (V1) - place in front of the 5th rib or over the pulse point behind and under the left front leg

RED (V3) - place between the 5th and 6th ribs directly behind the **BLACK** electrode patch and lead

PLEASE NOTE:

Dogs have 13 ribs. Begin counting backwards from the last rib (found in the loin) to determine each electrode's placement.

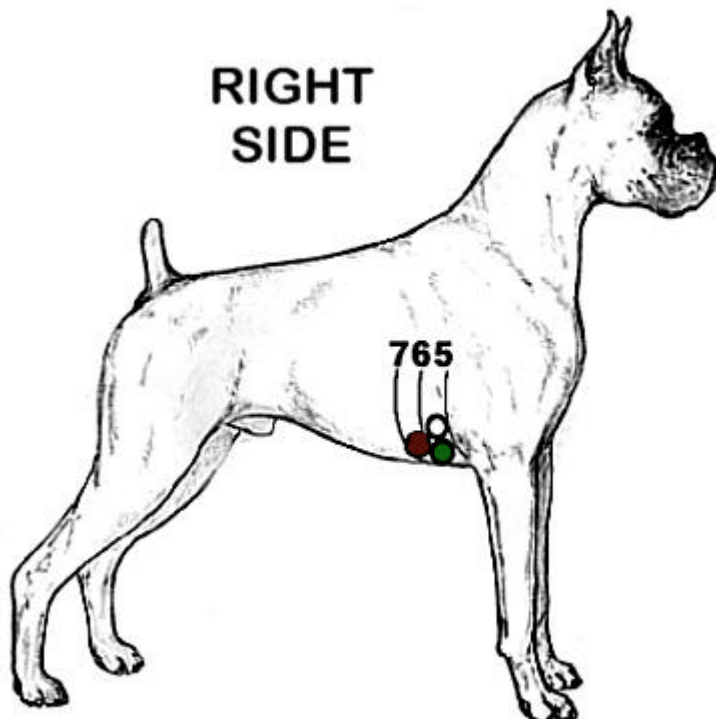

GREEN (ground) - place between the 5th and 6th ribs

WHITE (V1) - place above the **GREEN** electrode patch and lead

BROWN (V3) - place between the 6th and 7th ribs, directly behind the **GREEN** electrode patch and lead