


# 5-Lead Holter Hook-Up

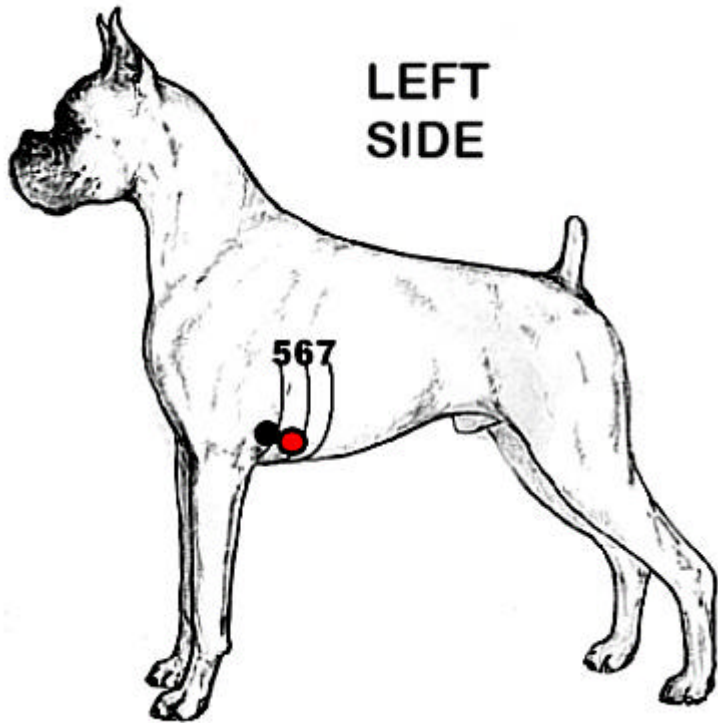


**BLACK (V1)** - place in front of the 5th rib or over the pulse point behind and under the left front leg

**RED (V3)** - place between the 5th and 6th ribs directly behind the **BLACK** electrode patch and lead

## PLEASE NOTE:

Dogs have 13 ribs. Begin counting backwards from the last rib (found in the loin) to determine each electrode's placement.

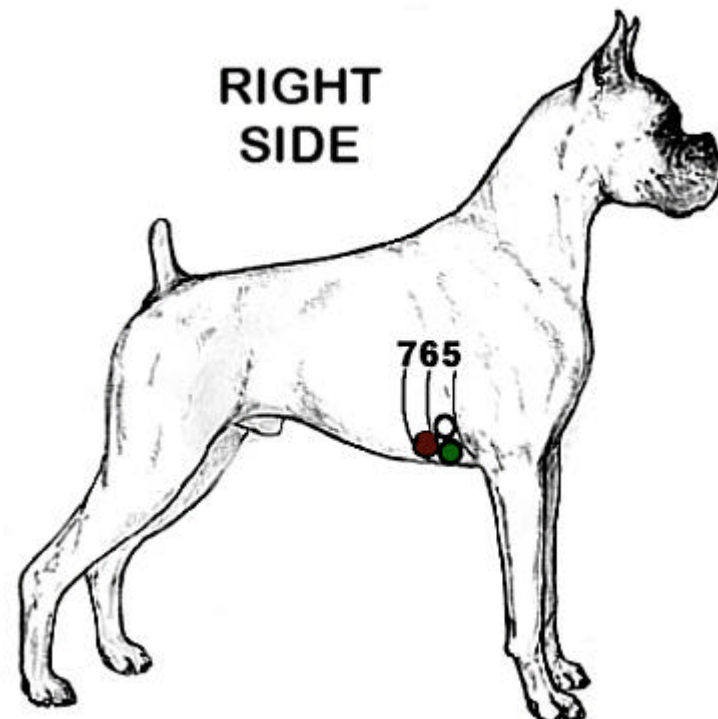

**GREEN (ground)** - place between the 5th and 6th ribs

**WHITE (V1)** - place above the **GREEN** electrode patch and lead

**BROWN (V3)** - place between the 6th and 7th ribs, directly behind the **GREEN** electrode patch and lead